



## Instructions for Use


1. Test Strips
2. Control Line
3. Test Line
4. Test Area
5. Test Legend
6. Specimen Cup
7. Spill-Free Cap

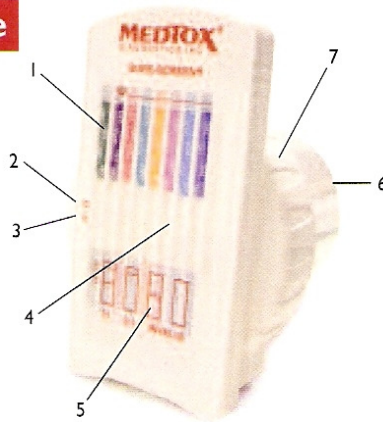


1. Donor voids into specimen cup. (A minimum of 45 ml)
2. Screw lid onto cup.

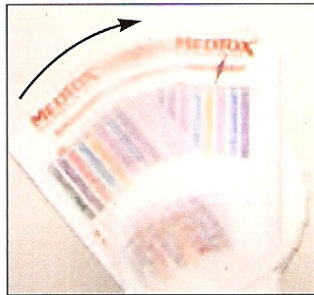


3. Remove Sure-Screen<sup>®</sup> device from foil pouch.
4. Place device onto lid and rotate clockwise until it snaps into place.

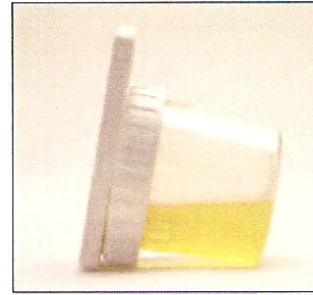

5. Place cup on its side as shown above to start urine flow. (If less than 45 ml of urine, you may need to tilt cup forward to begin flow)
6. Negative results can be read after line appears. Non-negative results can be read after five minutes.

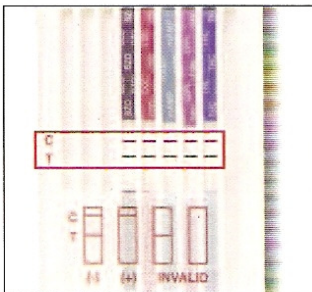

7. A line must appear in the "C" area in order for the test to be valid.
8. Any line in the test area is to be interpreted as negative.

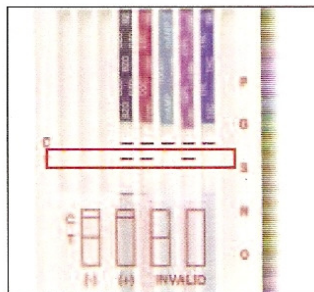

9. The absence of a line in the test area is to be interpreted as a non-negative.

### NOTE:

Line intensity may vary. The presence of any line, whether faint or broken, is to be interpreted as a negative result.

Technical Support: 1-877-643-5703